Warranty Card
This instrument is covered by a 2 year (mattress is 1 year) guarantee from the date of purchase, and accessories are not included. The guarantee is valid only on presentation of the warranty completed or stamped by the seller/dealer confirming date of purchase or the receipt. Opening or altering the instrument invalidates the guarantee. The guarantee does not cover damage, accidents or non-compliance with the instruction manual. Please contact your local seller/dealer or buying source or www.rossmax.com.
Customer Name: _____
Address: _____
Telephone: _____ E-mail address: _____
Gender: Male <input type="checkbox"/> Female <input type="checkbox"/> Age: _____
Product Information
Date of purchase: _____
Store where purchased: _____

Rossmax Swiss GmbH,
Tramstrasse 16, CH-9442 Berneck,
Switzerland

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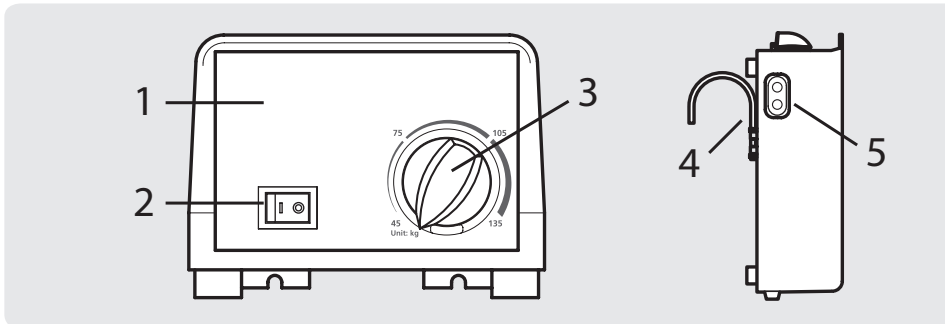
EN English

Introduction

Thank you for purchasing Rossmax cell air mattress. With proper care and use, your air mattress will provide you with many years of reliable treatments. We encourage you to thoroughly read this guide-book to learn about setting up and the features of your air mattress. The mattress relieves pressure by sequentially deflating and inflating alternate air cells on approximately 12 minutes timed intervals. It is intended to help and reduce the incidence of pressure ulcers while optimizing patient comfort, long term care of patients suffering from pressure ulcers.

Attention: Consult the accompanying documents. Please read this manual carefully before use. Please be sure to keep this manual.

Product caution

- Do not smoke on or near the pump.
- Keep the pump away from heated surfaces.
- Explosion risk if used in the presence of flammable anesthetics.
- Never operate this product if it has a damaged cord or plug, if it is not working properly, please contact Rossmax service if necessary.
- Replace fuse as marked: T1A 250V.
- Do not use with patients who have a spinal cord injury.
- Close supervision is necessary when this product is used on or near children.
- Use this product only for its intended use as described in this manual. Do not use other mattress not recommended by manufacturer.
- Do not modify this product without authorization of the manufacturer.
- If you suspect that you may have had or are having an allergic reaction, please consult your physician immediately.

How to use

- Place the mattress on the bed frame with the hose end at the foot section of the bed frame, Secure the mattress firmly by fixing the flags to the bed frame if available.
- Using the integrated hangers, securely hang the pump on the bed end at the foot end or place on a smooth flat surface.
- Connect the air hoses from the mattress to the pump unit.
Note: Check and ensure the air hoses are not kinked or tucked under mattress.
- Plug the pump into a wall electrical outlet. Be sure the power cord is safely away from possible hazards.
Note: After installation, the extra length of power cord, if any, should be neatly arranged to avoid any tripping accidents.
- Turn on the power switch on the control panel of the pump. The pump will begin to inflate the mattress.
Note: The pump should only be used with mattress recommended by manufacturer. Do not use it for any other purpose.
- While inflating, adjust the pressure setting based upon patient comfort level by turning the pressure adjustment knob clockwise to increase firmness.
Note: Every time the mattress is set up for use, it is recommended that the pressure first to be set to max., and then adjust air mattress weight levels to the desired softness after set up has been completed.

Clean and disinfection

- Wipe the pump unit with a damp cloth and mild detergent. If other detergent is used, choose one that will have no chemical effects on the surface of the plastic case of the pump unit.
- Wipe down the mattress with warm water containing a mild detergent and air dried thoroughly before use.
Note:
 - Do not use phenolic based product for cleaning.
 - After cleaning, dry the mattress without direct exposure sunlight.

Troubleshooting

Symptoms	Check point/Correction
Power is not ON	<ul style="list-style-type: none">Check if the plug is connected to mains. Check for a blown fuse.
Patient is bottoming out	<ul style="list-style-type: none">Pressure setting might be inadequate for the patient, adjust higher pressure and wait for a few minutes for best comfort.
No air produced from some air outlets of the air tube connector	This is normal since there is alternating mode, air outlets take turns to produce air during their cycle time.

Note: If the pressure level is consistently low, check for any leakage (tubes or air hoses). If necessary, replace any damaged tube or hoses or contact Rossmax service if necessary.

Specification

Pump unit	
Dimension (L x W x H)	26.5 x 13.5 x 10 cm
Weight	3 lbs (1.4 kg)
AC power source	AC 230V/50Hz or AC 220V/60Hz or AC 110V/60Hz
Cycle time	12 mins
Mattress	
Dimension (L x W x H)	79”x33.86”x3.74” (200 x86 x9.5 cm)
Weight	10 lbs (4.5 kg)
Weight Capacity	298 lbs (135 kg)
Operating Condition	10°C to 40°C (50°F to 104°F), 10% to 75% RH
Storage & Transportation Condition	-18°C to 43°C (0°F to 110°F), 10% to 95% RH
 IP Classification	Type BF applied parts <p>IP21: Protection against harmful ingress of water and particulate matter</p>

EMC guidance and manufacturer's declaration			
Guidance and manufacturer's declaration- electromagnetic emissions			
The AMA0 is intended for use in the electromagnetic environment specified below. The customer or the user of the AMA0 should assure that it is used in such an environment.			
Emission test			
RF emissions (CSPR 11)	Group 1	The AMA0 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.	
RF emissions (CSPR 11)	Class B	The AMA0 is suitable for use in all establishments, including domestic establishments and those directly connected to the public	
Harmonic emissions IEC 61000-3-2	Class A	low-voltage power supply network that supplies buildings used for domestic purposes.	
Voltage fluctuations/ricker emissions IEC 61000-3-3	Compliance		
Guidance and manufacturer's declaration-electromagnetic immunity			
The AMA0 is intended for use in the electromagnetic environment specified below. The customer or the user of the AMA0 should assure that it is used in such an environment.			
Immunity test			
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact <p>± 8 kV air</p>	± 6 kV contact <p>± 8 kV air</p>	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%
Electrical fast transient/burst IEC 61000-4-4	± 2kV for power supply lines	± 2kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1kV (line(s) to line(s)) <p>± 2kV line(s) to earth</p>	± 1kV differential mode <p>± 2kV common mode</p>	Mains power quality should be that of a typical commercial or hospital environment.
Voltage Dips, short interruptions and voltage variations 40% UT(160% dip in UT) for 5 cycles <p>70% UT(30% dip in UT) for 25 cycles</p> <p>power supply input lines IEC 61000-4-11</p>	<5% UT(>95% dip in UT) for 0.5 cycle <p>40% UT(60% dip in UT) for 5 cycles</p> <p>70% UT(30% dip in UT) for 25 cycles</p> <p><5% UT(>95% dip in UT) for 5 s</p>	<5% UT(>95% dip in UT) for 0.5 cycle <p>40% UT(60% dip in UT) for 5 cycles</p> <p>70% UT(30% dip in UT) for 25 cycles</p> <p><5% UT(>95% dip in UT) for 5 s</p>	Mains power quality should be that of a typical commercial or hospital environment. If the user of the AMA0 requires continued operation during power mains interruptions, it is recommended that the AMA0 be powered from an uninterruptible power supply or a battery.

EN Name/Function of each parts

- Control Panel
- ON/OFF Switch
- Pressure Adjust Knob
- Hanger
- Air Hose Port

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Power frequency (50/60 Hz)	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristics of a typical location in a typical commercial or hospital environment.
NOTE: UT is the a.c. mains voltage prior to application of the test level.			
Guidance and manufacturer's declaration- electromagnetic immunity			
The AMA0 is intended for use in the electromagnetic environment specified below. The customer or the user of the AMA0 should assure that it is used in such an environment.			
Immunity test IEC 60601 level) Compliance level) Electromagnetic environment- guidance			
Conducted RF IEC 61000-4-6	3 Vrms	3 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the AMA0 including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: d= 1,2 √P, d = 1,2 √P 80MHz to 800 MHz; d = 2,3 √P 800MHz to 2,5 GHz
Radiated RF IEC 61000-4-3	3 V/m	3 V/m	Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range. b. Interference may occur in the vicinity of equipment marked with the following symbol:
NOTE 1: At 80 MHz and 800 MHz, the higher frequency range applies. NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the AMA0 is used exceeds the applicable RF compliance level above, the AMA0 should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the AMA0.			
b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.			

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Recommended separation distance between portable and mobile RF communications equipment and the AMA0			
The AMA0 is intended for use in an electromagnetic environment in which related RF disturbances are controlled. The customer or the user of the AMA0 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the AMA0 as recommended below, according to the maximum output power of the communications equipment.			
Rated maximum output power of transmit- ter / PWR			
Separation distance according to frequency of transmitter / m			
	150 kHz to 80 MHz / d=1,2√P	80 MHz to 800 MHz / d=1,2√P	800 MHz to 2,5 GHz / d=2,3√P
0,01	0,12	0,12	0,23
0,1	0,38	0,38	0,73
1	1,2	1,2	2,3
10	3,8	3,8	7,3
100	12	12	23
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.			
NOTE 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.			
NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

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