


## Vaš uTV™ 3 na prvi pogled

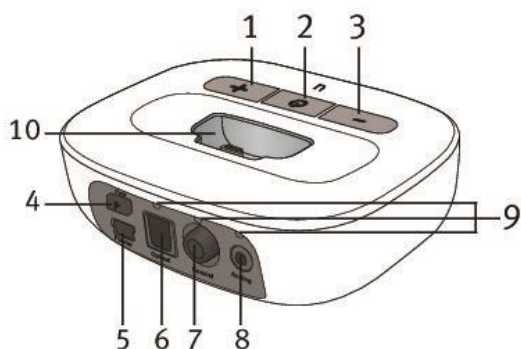

- 1 Taster za pojačavanje
- 2 Taster za uklj./isklj. i glavna signalna lampica
- 3 Taster za utišavanje
- 4 Taster za uparivanje
- 5 Napajanje
- 6 Digitalni audio (optički)
- 7 Digitalni audio (koaksijalni)
- 8 Analogni audio (3.5 mm)
- 9 Audio signalne lampice
- 10 Izmenljivi otvor za punjenje

## Značenje signalnih lampica

|                              |                        |  |
|------------------------------|------------------------|--|
| Uključen                     | Svetli zeleno          |  |
| Uključen ali bez audio ulaza | Kratko trepćuće zeleno |  |
| Režim uparivanja             | Brzo trepćuće plavo    |  |
| Režim prenosa                | Svetli plavo           |  |
| Režim pauze                  | Kratko trepćuće plavo  |  |
| Režim testiranja             | Trepćuće u više boja   |  |

# uTV3 kratko uputstvo za upotrebu



Unitron Hearing  
20 Beasley Drive, P.O. Box 9017,  
Kitchener, ON N2G 4X1 Canadá




Unitron Hearing GmbH  
Max-Eyth-Straße 20, 70736  
Fellbach-Oeffingen, Germany


[unitron.com](http://unitron.com)



# Korak 1

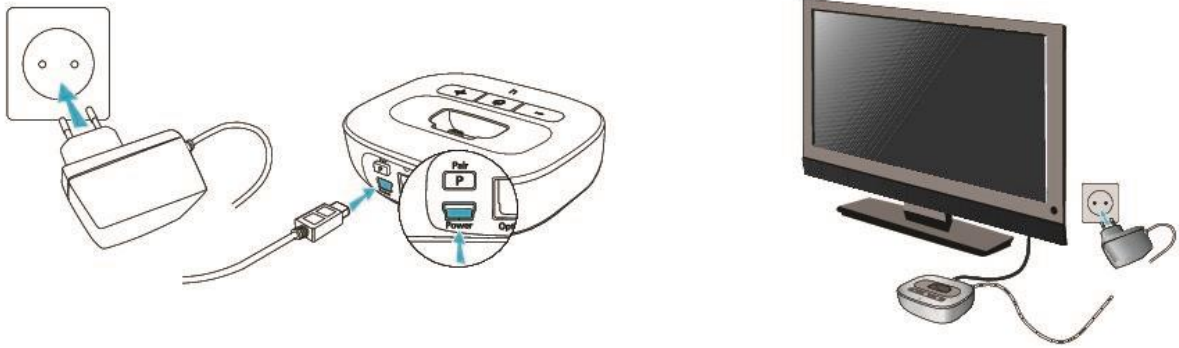

# Korak 2

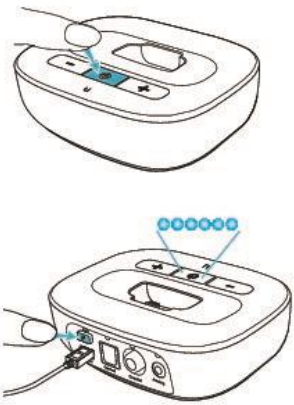


# Korak 3


uDirect 3



uStream

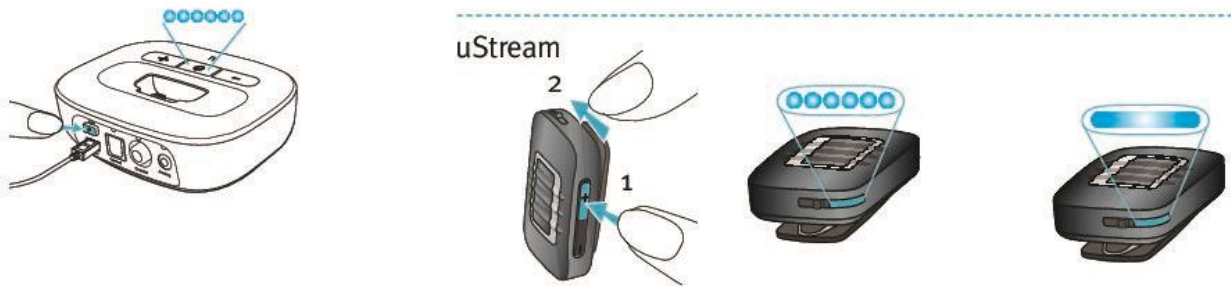

# Korak 4

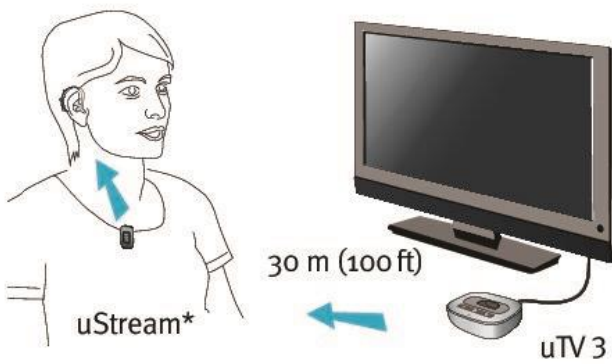

\*uDirect 3 je takođe kompatibilan.

# Korak 5

